# CleanSafe basic

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**Installation:** It is the installer's responsibility to ensure installation of unit complies with all applicable codes, standards and regulatory requirements. Do not install this dispenser above heat sources or near sources of ignition/flame. **Intended use:** The device is intended solely for use with disposable cartridges.

---

**BODE Chemie GmbH,**  
Melanchthonstraße 27, 22525 Hamburg, Germany, Tel. +49 40 54 006-0.

---

© 1 L dispenser, plastic, manual

## For hand disinfectants, wash and skin care lotions

### 1. Scope of delivery

Dispenser, fixing set, 2 keys, instruction manual.

### 2. Opening the dispenser

Open dispenser with one of the enclosed keys and open the dispenser housing to the front. The key (round) can be removed from the dispenser in order to lock the dispenser securely. The permanent key (rectangular) remains after the first use in the dispenser and allows a fast opening without a key.

### 3. Removal of backplate

Press the latch on the backplate of the dispenser and slide the dispenser upwards, off the plate.

### 4. Installation advice

Dispenser should be installed at least 8 inches (20.5 cm) from any elevated surfaces. The most ergonomically-suitable installation height for safe and comfortable operation is 110 cm, measured from the floor to the base of the dispenser housing.

### 5. Mounting of backplate

*For screwing solution:* Attach the backplate with enclosed attachment materials to the wall. Please check suitability of wall beforehand! *For adhesive solution:* Clean the surface with the supplied alcohol swab and wait for 5 minutes. Attach the backplate with the adhesive pads on to the wall. Wait 24 hours before installing the dispenser.

### 6. Mounting of dispenser

Slide the dispenser from above on to the mounting plate till the locking latch snaps in.

### 7. Bottle Placement/Exchange

Remove cap of pump. Insert filled bottle from the front into the dispenser until the bottle is securely locked at pump thread. *Attention!* For insertion pump must be in its original state (not pulled out). For bottle exchange pull the bottle out of the dispenser towards the front.

### 8. Dispenser operation

Close the dispenser housing. Hold your hand under the nozzle and operate the lever several times until solution is dispensed.

## Cautions

**Cleaning and Maintenance:** Clean exterior surfaces of the dispenser with damp cloth. Do not use acid (chloride) or abrasive cleaners. Dispenser is not autoclavable and cannot be cleaned in the dishwasher.