CleanSafe extra touchless

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© 1 L dispenser, metal, touchless

For hand disinfectants, wash and skin care lotions

1. Scope of delivery

Dispenser, fixing set, key, 4 batteries, instruction manual.

2. Opening the dispenser

Open dispenser with the enclosed key. Push the dispenser housing with both hands vertically upwards until you notice resistance and open it to the front.

3. Installation advice

Dispenser should be installed at least 8 inches (20.5 cm) from any elevated surfaces. The most ergonomically-suitable installation height for safe and comfortable operation is 110 cm, measured from the floor to the base of the dispenser housing.

4. Mounting of dispenser

Attach the dispenser with enclosed attachment materials (4 screws and wall plugs) to the wall. Please check suitability of wall beforehand!

5. Insert the batteries into the dispenser.

Open battery cover and insert the batteries. Please pay attention to the correct polarity. Close battery cover. Exclusively operate the dispenser with leakproof alkaline batteries of the same manufacturer and size type C with the same charging condition. Do not use the unit with rechargeable batteries.

6. Bottle Placement/Exchange

Remove cap of pump. Insert filled bottle from the front into the dispenser until the bottle is securely locked at pump thread. *Attention!* For insertion pump must be in its original state (not pulled out). For bottle exchange pull the bottle out of the dispenser towards the front.

7. Dispenser operation

To close the dispenser housing, lift it gently in order to lift it out of the safety catch. Lock the dispenser with the key. Hold your hand under the nozzle and operate the dispenser several times until solution is dispensed. If light illuminates green on actuation of dispenser (batteries are OK) the dispenser is ready for operation. If light shines red dispenser is not closed properly. If the dispenser does not operate as normally expected, resolve it by ensuring that new batteries have been inserted into the device.

8. Stroke selection

The dispensers' standard operating mode is one stroke per actuation. By using the selector switch the operating mode can be adjusted to 2 or 3 strokes per actuation.

Cautions

Cleaning and Maintenance: Clean exterior surfaces of the dispenser with damp cloth. Please use only cleaning agents suitable for stainless steel surfaces. Do not use acid (chloride) or abrasive cleaners. Dispenser is not autoclavable and cannot be cleaned in the dishwasher. **Installation:** It is the installer's responsibility to ensure installation of unit complies with all applicable codes, standards and regulatory requirements. Do not install this dispenser above heat sources or near sources of ignition/flame. **Intended use:** The device is intended solely for use with disposable cartridges. May only be used in enclosed spaces, in non-explosive and non-condensing atmospheres. Not in swimming baths. **Disposal of used electrical and electronic devices:** The symbol on the product or its packaging indicates that this product should not be treated as standard household waste, but should be taken to a recycling point for electrical and electronic devices. Further information may be obtained from your municipality, the municipal disposal companies or the company from where you purchased the product. **Statutory information on battery disposal:** Batteries should not be disposed of as household waste. As a consumer, you are statutorily obliged to return used batteries. Old batteries should be surrendered to the public collection points in your municipality or to anywhere where batteries of this type are sold. **Conformity:** EMC directive 2014/30/EU, ROHS 2011/65/EU, WEEE 2012/19/EU 

BODE Chemie GmbH,

Melanchthonstraße 27, 22525 Hamburg, Germany, Tel. +49 40 54 006-0.