Veroval[®]

Veroval®

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The screenshot displays the Veroval® medi.connect web interface. At the top, the logo 'Veroval® medi.connect' is shown with a colorful bar below it. A navigation menu includes 'Panoramica', 'Pressione arteriosa', 'ECG', 'Peso', 'Temperatura', and 'Self-test'. The 'Pressione arteriosa' section is active, showing a 'vista' button and icons for a line graph, a circular refresh icon, a list view, and a PDF icon. Below this, there are date selection fields: 'Settimana', 'Mese', and 'selezionare' followed by two date pickers showing '22/07/2017' and '23/07/2017'. A 'Ricerca' button and a 'MAD' checkbox are also present. At the bottom of the date pickers, there are radio buttons for 'mattina', 'sera', and a checked 'giornata intera' option. A black mouse cursor is pointing at the calendar icon of the first date picker.

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The screenshot displays the Veroval® medi.connect web interface. At the top, the user is identified as Mario Rossi (M1 M2). The main navigation bar includes: Panoramica, Pressione arteriosa (selected), ECG, Peso, Temperatura, Self-test, and Impostazioni. The 'Pressione arteriosa' section is active, showing a 'vista' menu with icons for line graph, clock, list, and PDF. Below this, there are date selection fields: 'Settimana' and 'Mese selezionare' with a calendar icon, followed by '22/07/2017' and '23/07/2017' with calendar icons, a 'Ricerca' button, and a 'MAD' checkbox. A calendar for 'luglio 2017' is open, showing days of the week (lu, ma, me, gi, ve, sa, do) and dates. The date '22' is highlighted in blue, and a mouse cursor is pointing at it. Other dates visible include 26, 27, 28, 29, 30, 1, 2, 3, 4, 5, 6, 7, 8, 9, 10, 11, 12, 13, 14, 15, 16, 17, 18, 19, 20, 21, 23, 24, 25, 26, 27, 28, 29, 30, 31.

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
The screenshot displays the Veroval® medi.connect web interface. At the top, the user is identified as Mario Rossi (M1 M2). The main navigation bar includes: Panoramica, Pressione arteriosa (selected), ECG, Peso, Temperatura, Self-test, and Impostazioni. The 'Pressione arteriosa' section is active, showing a 'vista' menu with icons for line graph, pie chart, list, and PDF. Below this, there are date selection fields: 'Settimana' and 'Mese selezionare' with a calendar icon, and a date range from 22/07/2017 to 23/07/2017. There are also checkboxes for 'mattina', 'sera', and 'giornata intera', and a 'Ricerca' button. A calendar for July 2017 is overlaid, with the date 23 highlighted. A mouse cursor is pointing at the 23rd. The calendar table is as follows:

luglio 2017						
lu	ma	me	gi	ve	sa	do
26	27	28	29	30	1	2
3	4	5	6	7	8	9
10	11	12	13	14	15	16
17	18	19	20	21	22	23
24	25	26	27	28	29	30
31	<	<	<	<	<	<

...al...

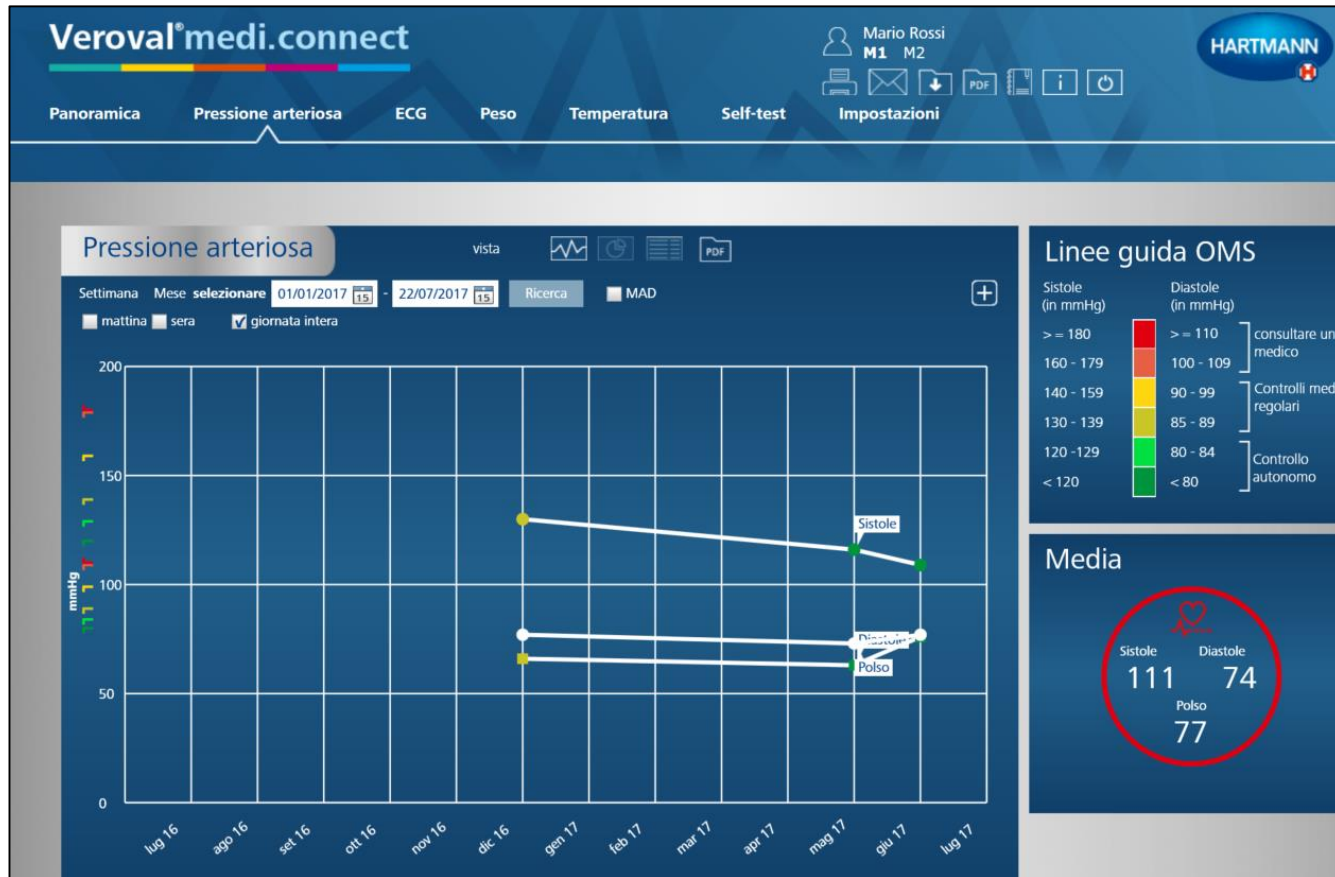The screenshot displays the Veroval® medi.connect web application interface. At the top, the logo 'Veroval® medi.connect' is visible, followed by a navigation bar with tabs for 'Panoramica', 'Pressione arteriosa', 'ECG', 'Peso', 'Temperatura', and 'Self-test'. The 'Pressione arteriosa' tab is active. Below the navigation bar, the main content area features a header for 'Pressione arteriosa' and a 'vista' section with icons for a line graph, a circular refresh icon, a list view, and a PDF icon. The search filters include 'Settimana', 'Mese', and 'selezionare' with date pickers for '22/07/2017' and '23/07/2017'. There are also checkboxes for 'mattina', 'sera', 'giornata intera' (checked), and 'MAD'. A prominent black mouse cursor is pointing at the 'Ricerca' button.

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