

ColActive® PLUS

Collagen Matrix Dressing



✓ FACILITATES NATURAL WOUND HEALING

EDTA and collagen in ColActive® Plus target and deactivate elevated MMP Activity*.

✓ MOIST WOUND ENVIRONMENT

Alginate and CMC in ColActive® Plus allow the dressing to absorb up to 20 times its weight in wound drainage, while still maintaining an optimal moisture balance for wound closure*.

✓ CLINICAL TRACK RECORD

ColActive® Plus has over ten years of clinical success in treating chronic stalled wounds.

HOW DOES IT WORK?

ColActive® Plus is an advanced wound dressing made from collagen, sodium alginate, carboxymethylcellulose (CMC), and ethylenediaminetetraacetic acid (EDTA). ColActive® Plus is a topically applied wound dressing that will transform into a soft gel sheet when in contact with wound exudates. The dressing maintains a moist wound environment that aids in the formation of granulation tissue and epithelialization. The EDTA in the dressing removes zinc to inhibit the activity of matrix metalloproteinases (MMP), thus creating a suitable environment for wound healing. ColActive® Plus dressings may be trimmed and layered for the management of deep wounds.

WHEN TO USE:

ColActive® Plus is indicated for management of full and partial thickness wounds including: pressure ulcers, diabetic ulcers, ulcers caused by mixed vascular aetiologies, venous ulcers, donor and graft sites, abrasions, traumatic wounds healing by secondary intention, dehisced surgical wounds, first and secondary degree burns.

WHEN NOT TO USE:

ColActive® Plus dressings should not be used on patients with a known allergy or sensitivity to animal derived collagen, any other ingredient or on third degree burns.

*Covalon Technologies Ltd., Data on file.

Manufactured by:

covalon
TECHNOLOGIES LTD.

www.covalon.com


Distributed by:



For ordering information call 800-243-2294
or visit www.hartmannusa.com

HOW TO APPLY:

1. Debride and irrigate the wound bed with an appropriate wound cleanser or distilled water before application.
2. Cut the dressing to fit the exact wound size. For heavily exudating wounds, apply the dressing directly to the wound bed. For dry wounds with very little exudate, moisten the wound bed with distilled water to begin the gelling process.
3. Cover the ColActive® Plus with an appropriate secondary dressing.
4. Change dressings daily or as good nursing practice dictates.

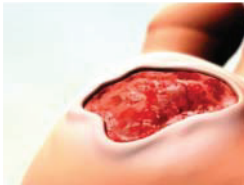

1.



2.



3.



4.

CARE AND REMOVAL:

1. Remove the secondary dressing with care. Gently remove ColActive® Plus ensuring that the dressing removal does not damage any newly formed tissues.
2. Use an appropriate wound cleanser or distilled water to cleanse the wound site prior to applying a new ColActive® Plus dressing, as per application instructions.

AVAILABILITY:

COLACTIVE® PLUS

Collagen Matrix Dressing

Reference Number	HCPCS	Size (in)	Dressings per Carton / Cartons per Case
1016 0000	A6021	2in x 2in	10/10
1018 0000	A6021	4in x 4in	10/10

NOTE:

- HCPCS codes contained in this document are provided for informational purposes only and represent no statement, promise, or guarantee by Covalon Technologies Ltd. concerning levels of reimbursement, payment, or charge.
- As with any medical device, always refer to the full instructions for use prior to application.

Manufactured by:

covalon
TECHNOLOGIES LTD.

www.covalon.com

Distributed by:



For ordering information call 800-243-2294
or visit www.hartmannusa.com