

USER GUIDE

Zetuvit® Plus Silicone Border is a self-adhesive superabsorbent dressing with silicone interface for the treatment of moderately to heavily exuding, chronic and acute wounds. The silicone layer in contact with the wound allows easy application and painless, atraumatic removal. The absorbent pad absorbs and retains the exudate.

Zetuvit® Plus Silicone Border removes excessive MMP's activity and exudate, prevents maceration and excoriation to the wound bed whilst maintaining optimal microclimate¹ for wound healing.

Where can Zetuvit® Plus Silicone Border be used?



Moderate exudate
Sloughy tissue
High exudate
Granulating tissue
Under compression



Dry Wounds
Necrotic Tissue

Where exudate is moderate to high



Acute wounds (traumatic wounds, burns, postoperative wounds)
Chronic wounds (ulcers, pressure sores, fungating wounds, diabetic wounds).



Small Sinuses
Small Cavaties

Applying Zetuvit® Plus Silicone Border

Choose the appropriate dressing size for the wound

Place directly onto the wound bed as a primary dressing



To ensure optimum results for Zetuvit® Plus Silicone Border follow the guidelines below:




- Use on moderate to highly exuding wounds
- Choose appropriate dressing size to fit the wound
- Apply directly to the wound bed as a primary dressing
- Use instead of silicone adhesive foams
- Use under compression and areas of pressure
- Examine the dressing regularly in line with current protocol
- Use on infected wounds as a complementary treatment regime as bacteria are sequestered and retained within the dressing
- Use as a protease modulator to assist with wound healing
- Use to reduce the risk of maceration to peri-wound tissue
- Leave in place for up to 3 days



- Do not use on necrotic wounds
- Do not use on bleeding wounds
- Do not cut the absorbent green pad (transparent silicone border is cuttable)
- Do not use Zetuvit® Plus Silicone Border with multiple absorbent dressings. Doing so may increase the cost of wound management, and is not required due to its absorbency and retention capacity

Product Composition

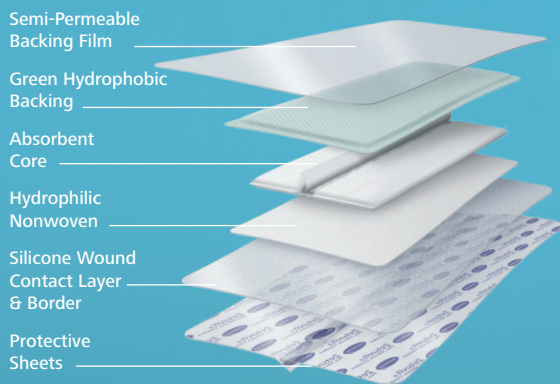

HARTMANN UK & Ireland
Unit P2 Parklands, Heywood
Distribution Park, Pilsworth Road,
Heywood OL10 2TT

UK: +44 (0) 1706 363200
Ireland: +353 (0) 1800 937766
Email: info@uk.hartmann.info
www.hartmann.co.uk

