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PL Opis produktu

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Product description

Zetuvit Plus Silicone is a dressing for the treatment of moderate to highly exuding, acute and chronic wounds.

The silicon layer in contact with the wound allows easy application and painless, atraumatic removal.

The absorbent pad absorbs and retains the exudate.

Composition
Zetuvit Plus Silicone comprises an absorbent pad made from cellulose and superabsorbent polyacrylate for the absorption and retention of exudate. On the wound side it is covered with a fine-mesh silicone interface to facilitate the application of the dressing and promote non-adherence to the wound. The backing of the dressing is made of green-coloured, polypropylene nonwoven, which is hydrophobic but permeable to air, allowing gas exchange.

Properties and mode of action

Zetuvit Plus Silicone absorbs exudate and retains it in the absorbent pad. The micro-adherent, silicon interface prevents the dressing from slipping and facilitates its application. The dressing changes are carried out atraumatically and painlessly.

Indications

Zetuvit Plus Silicone is suitable for the treatment of wounds with moderate to high levels of exudate: acute wounds (traumatic wounds, burns, post-operative wounds) and chronic wounds (ulcers, pressure sores, tumor wounds, diabetic feet).

Mode of application and dressing change

- Clean the wound using a physiological solution or as instructed by a clinician.
- Choose a suitable dressing size for the wound making sure that the central pad exceeds the edges of the wound by around 1 to 2 cm. Do not cut the dressing.
- Before applying the dressing, after rinsing the wound, make sure that the area around the wound is dry using swabs to ensure that the dressing adheres.
- Remove the protective sheets. Place the dressing with the adherent side onto the wound, so that the green side faces away from the wound.
- Secure the dressing using a Peha-haft bandage or any other suitable fixation, or, if need be, with compression bandages as instructed by a clinician.
- The dressing can remain on the wound for several days depending on the condition of the wound and the established clinical protocol.
- The dressing must be changed if clinically indicated or when exudate reaches the rim of the dressing.

Special precautions

- Do not cut the dressing.
- Before treating a wound with healing impairment a clinician has to be consulted to establish a clinical diagnosis. Treatment with Zetuvit Plus Silicone cannot replace a causal treatment of the wound-healing impairment. If there are clinical signs of infection, the infection needs to be controlled with appropriate treatment before this dressing can be used.
- In all cases, follow the established clinical protocol.
- In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution following a clinician's recommendation.

Contraindication
Hypersensitivity to any of its components.

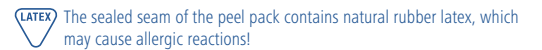
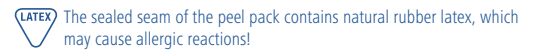
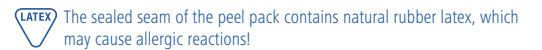
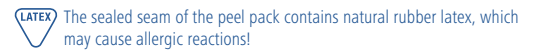
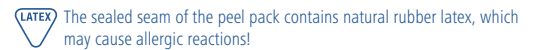
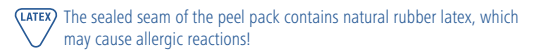
Side effects
Increased exudate, maceration, increased pain, adherence of the dressing to the wound, itching, burning, oedema, multiple ulceration, skin torn, blisters, erythema. If any of these side effects occur, consult a clinician.

Special notes

Store flat in a dry place, protected from light. Do not resterilize.

Date of revision of the text: 2017-07

-  **Do not use if package is damaged**

-  **STERILEEO**     
- The sealed seam of the peel pack contains natural rubber latex, which may cause allergic reactions!**

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

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